

Operating instructions for *LEDental Loupe 6.0x*


1. Security measures

Read these safety precautions carefully before use and operate the unit correctly.

These instruction symbols enable you to use this product safely and avoid danger and harm to yourself and others. They are classified according to the extent and severity of the hazard or harm. All instructions and symbols concern safety and must be observed.



This sign warns of an extremely dangerous situation where failure to observe the danger notice will result in death or serious irreversible injury.

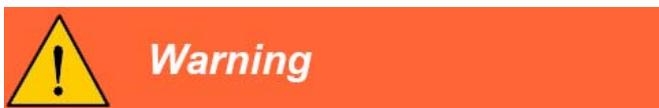

This sign warns of a dangerous situation where failure to observe the danger notice may result in death or serious irreversible injury.



This sign warns of a dangerous situation where failure to observe the danger notice may result in minor reversible injury.




This sign warns of situations where failure to observe the notice may result in damage to the unit or malfunction.

2. Adjustment

The magnifier is attached to the headband, double frame or sporty frame with two screws (1).
With the double joint (2), the optics can be optimally adjusted to the eyes.

To adjust the pupil distance, the two optics can be moved separately. To do this, first loosen the adjustment screws (3). After the adjustment, they must be fixed again.

The PowerLight spot is attached to the frame with the help of the metal clip (4). To do this, loosen the small screw (5) until the clip can slide onto the frame. Then tighten the screw again.

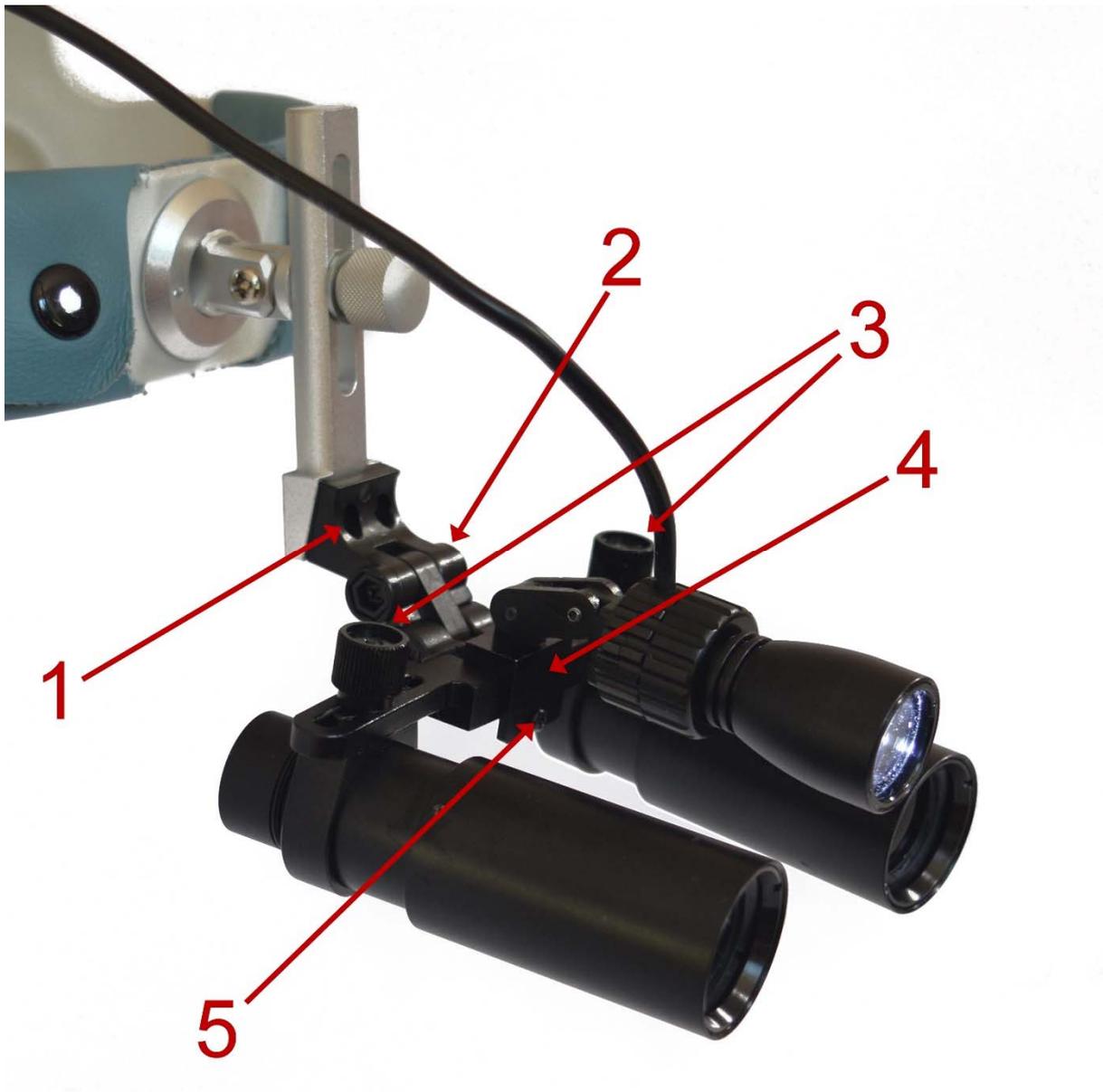

3. Technical data

LEDental Loupe 6.0x

Magnification factor: 6.0

Weight: 85 g

Working distance:

6.0x-S: 410 mm

6.0x-R: 490 mm

Diameter field of view:

6.0x-S: 55 mm

6.0x-R: 58 mm

LEDental GmbH
Mahlower Strasse 28
15806 Zossen
Germany
Phone: +49 (3377) 20 41 39
Fax: +49 (3377) 200 79 65
mail@ledental.de
www.ledental.eu

